

INSTRUCTIONS FOR USE


DTFM-100

DIGITAL THERMOMETER

FEATURES

1. LCD display (Liquid Crystal Display)
2. Flexible tip (to avoid breakage)
3. Dual Mode C & F
4. Auto turn off

COMPONENTS

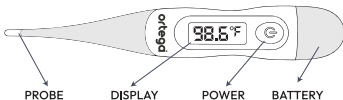

SPECIFICATIONS

Range: 32°C -42.9°C / 89.6°F - 109.22°F

Accuracy: ±0.2°C / ±0.4°F

Display: Liquid Crystal Display 3½ digits

Battery: One DC 1.5V Battery - LR41

Weight: Approximately 10g (Including battery)

Dimension: 9.5mm x 17.5mm x 128.5mm

Alarm: Beep sound when reading is complete.

BATTERY REPLACEMENT

1. When the battery is depleted, the display will indicate the following symbol ""
2. Remove the battery compartment as illustrated in the component section above.
3. Use an insulated tool to remove the battery, avoiding any sharp metal objects.
4. Insert the replacement battery & close the cover.
5. Ensure the battery polarity is positioned correctly.

WARRANTY

This thermometer is covered by a 1-year warranty from the date of purchase against manufacturing and performance defects. The warranty is valid only when the original purchase tax invoice is provided, showing the invoice number, model number & date of purchase.

STORAGE CONDITIONS

Store the thermometer in a clean, dry place away from direct sunlight, high temperatures, excessive humidity, and dust. Do not drop the thermometer or subject it to heavy shock. If the thermometer has been kept in a cold area, place it in a warm area before use. Clean the thermometer by wiping it with a soft cloth moistened with disinfectant, such as ethyl alcohol.

USAGE GUIDELINES

Turn on the thermometer using power button

Oral: Place the probe tip under the tongue and keep the mouth closed during measurement.

Armpit: Place the probe tip in the center of the armpit and hold the arm firmly against the body.

Rectal: Gently insert the probe tip into the rectum and do not force it during use.

Keep the thermometer steady until the measurement is complete or until you hear the beep sound.

WARNING

1. Do not attempt to disassemble, modify, or repair the thermometer, as this may affect accuracy.
2. Do not use in areas with strong electromagnetic interference, as this may cause incorrect readings.
3. Dispose of used batteries properly. Do not place batteries in fire, as they may explode.

CAUTION

Use only for body temperature measurement in the mouth, rectum, or armpit, under adult supervision. Keep out of reach of children.